

YD MAGIC MIRROR MANUAL INSTRUCTION

Please read the instruction carefully before usage

Thanks for purchasing YD MAGIC MIRROR PMU Machine. It is a pmu kit specially designed for starters as well as trainers. We hope you will enjoy its use and reliability. During your usage, if you need any service, please contact the representative or distributor from whom it was purchased. In order to avoid any unnecessary inconvenience on your part, we suggest reading these instructions carefully before contacting our authorized service representative.

Unpacking:

Check that you have received the following items with MAGIC MIRROR.

- MAGIC MIRROR Power Supply
- Adapter and power cord
- Hand piece and connect cord
- Kit case
- 1RL 10pcs



Cleaning:

Clean the control panel with soft cloth which has lightly moistened with mild detergent solution. Do not use any type of abrasive pad, sourcing power or solvent such as alcohol. Clean the outside of the control unit with a soft cloth using a disinfectant.

Safety:

To prevent fire or shock hazard, do not expose the unit to rain or moisture. To avoid electrical shock, do not open the cabinet. Refer servicing to qualified personnel only. Do not place MAGIC MIRROR near heat sources or in a place subject to direct sunlight, excessive dust or mechanical shock.

Should any solid object or liquid or liquid fall into the cabinet, unplug the receiver and have it checked by qualified personnel before operating it any further.

If you have any questions or problems concerning your device, please contact your dealer.

Power sources:

- Before operating the Device check that the operating voltage is identical to your local power supply. The operating voltage is indicated on the name plate at the rear of the control unit.
- The unit is not disconnected from AC power source (mains) as long as it is connected to the wall outlet, even if the unit itself has been turned off.
- If you are not going to use the Device be sure to disconnect it from the wall outlet. To disconnect the AC power cord, grasp the plug itself, never pull the cord.
- The AC power cord must be changed only at the qualified representative.

MAGIC MIRROR PMU Device was manufactured according to the VDE - guidelines and left the manufacturing facility in correct working condition. The device can only be operated with the power supply unit listed on the type indication plate. The device may be opened for repair and maintenance only by the manufacturing firm or designated authorized person. The device is designed for indoor operation. It should be operated at a room temperature ranging between 10°C and 35°C. The power source voltage has to correspond to the voltage of the device indicated on the back of the control unit!

If it seems that the device can not be operated without potential danger, switch it off and protect it from unintentional and unauthorized use.

Safe operation is not guaranteed:

- if the device or accessories show visible damage
- if the device does not work correctly
- after long storage at bad conditions
- after serious transport stress

In case any of the before mentioned occurs, the device should be returned to the manufacturer or an authorized distributor for technical inspection. This device should be operated by trained and authorized personnel only. The device should be placed on a solid and even base at all times.

Operation of MAGIC MIRROR PMU Machine

Connect power supply

On the bottom of the control panel, there is a socket that connects the adapter cord.

Connect the hand piece

Connect the hand piece to MAGIC MIRROR by connect cord, can be connected to socket. The socket is located on the bottom of MAGIC MIRROR.

Inserting the needle cartridge into the hand piece

Hold the hand piece with the insertion opening facing upwards. Screw the needle cartridge into the receiver opening in the hand piece. With the hand piece pointed upwards, until you feel it screwed in place, make sure the cartridge is locked in tight. If you're not sure that it's locked in place, hold the hand piece up by the cartridge. If it's not locked in the hand piece, the hand piece will be loosen from the needle cartridge.

Turn the power on

With all the steps before mentioned completed, you can now turn on the control unit. With the hand piece connected to socket, press the "ON/OFF" button, the light will appear.

Choose "AMTS,BROW,LINE,LIP" by press the "□" button as your need. Adjust the needle speed according to your procedure by "UP" or "DOWN".

Operating the device

By pressing the "POWER" on the device, the hand piece will start working by itself.

Pressing the "POWER" again, the hand piece will stop working.

Adjust penetration depth

You are now in a position to adjust the penetration depth of the needle. The handle of the hand piece serves as an adjustment ring. By turning the adjustment ring on the hand piece to the right, the penetration depth can be increased. Please note that the needle should never be adjusted to the highest setting. If the hand piece is not running perhaps the adjustable ring may have been over tightened.

During the procedure, check the needle depth frequently, since when moving from your client to the pigment cup, you may be inadvertently turning the adjustment ring on the hand piece.

Choose your favourite speed

The LED in the middle of the control unit displays the programme function e.g. brows, line, Lips, AMTS and sets the needle per second rate (NPS).to personal preference. After choosing the program function, by pressing the "UP" or "DOWN" keys on the control unit, it can be adjusted between 100 up to 900.

Needle Sizes and suggested needle speeds

(Speeds will vary per skin type and needle size)

Procedure	Needles Commendation	Speed
BROW HAIRSTROKE	1R 0.18mm,0.25mm,0.30mm,3F, 11U	400~650
BROW SHADING	1R 0.40mm,0.5mm, 3RS, 5RS	400~650
LINE	1R 0.30mm, 3RL	650~800
LIP CONTOUR	1R 0.30mm	500~900
LIP SHADING	5RS,5F,7F	500~900
AMTS	#12 microneedle	650~800

Safety cartridge

Attention - risk of injury! The cartridge contains needles that can come out if handled improperly and may lead to injuries. Avoid pressure on the needle shaft (back of the cartridge) since it forces the needle to exit the needle tube. The cartridges are sterilized by the manufacturer and delivered in a sterile packaging.

They are designed for single use and to be taken out of the sterile packaging immediately before use. When inserting the cartridges one must not touch them on the tip (area that comes in contact with the patient).

During work one has also to take care that areas of the cartridge that get into touch with the patient do not come in contact with contaminated items, for instance clothes.

In this case one has to use a new sterile cartridge. After the treatment cartridges must be disposed of into a safety box for clinical waste.

On each package containing safety cartridges is a label indicating lot number, production and expiry date. The lot number serves the explicit identification of the batch and has to be submitted to the manufacturer in case of problems. There must be guaranteed and not damaged classification and back tracing for all cartridges in the package; this means that the safety cartridge with the label has to be used last.

The delivered cartridges are sterile. Don't use cartridges from a damaged packaging. The manufacturer guarantees sterility for a period of 5 years if the packaging is closed.

Technical parameters

Type	MAGIC MIRROR
Nominal voltage	5V~12V
Frequency	50/60 Hz
Power input	100-240V 0.3Amax
Power output	5V---1000mA
Working frequency	100~900
Drive	Precision ubs adapter
Protecting class	protection small voltage
Operating mode	Continuous operation
Operating conditions	●Ambient temperature: + 10°C up to + 35°C ●Relative humidity: 30% up to 75%

This instrument has been tested and is in compliance with CE, FCC,SAA design and specifications are subject to change without notice.

Conditions for transportation and storage

Ambient temperature: + 5°C up to + 40°C

Relative humidity: 30% up to 75%

Air pressure: 900 up to 1060 hPa

Non-liability

- Only authorized trained medical specialized person or certified trained technicians are allowed to use the device and only for the purpose mentioned in the operating instructions.
- In the case of any and all severe infectious diseases (AIDS, hepatitis etc.) and hemophilia, use of the device can lead to contraindications in those clients!
- The user is obliged to carry out the principles of sterilization and sanitation and to work with sterile/disinfected instruments and materials only.
- . He/she should wear latex, or comparable gloves and ask the client for a medical clearance if necessary.
- . MAGIC MIRROR does not assume liability for deviate putting into operation or deviate application of this device nor as a result of it.

Warranty

MAGIC MIRROR guarantees 1 year on the hand piece and control panel from the time of its original purchase.